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PLEASE READ THESE INSTRUCTIONS CAREFULLY AND KEEP THEM

### Hand-wrist-forearm brace

#### TRADEMARKS OF MATERIALS

Velcro® is a registered trademark of Velcro Industries B.V.

#### SAFETY PRECAUTIONS

We recommend that the pressure exerted by the device does not act on parts of the body with wounds, swelling, or welts. It is advisable to not over tighten the device so as to avoid generating areas of excessive local pressure or the compression of underlying nerves and/ or blood vessels. Furthermore, we recommend wearing a garment to avoid direct contact with the skin. If in doubt on how to apply the device, contact a doctor, physiotherapist, or an orthopedic technician. Read the product composition on the internal label carefully. We recommend to not wear the device in the proximity of free flames. Do not apply in direct contact with open wounds.

#### WARNINGS

It is advisable that the device, designed for the specific indications as set out below, be prescribed by a doctor or a physiotherapist and applied by an orthopedic technician, according to the specific needs of the patient. To ensure effectiveness, tolerability, and proper functioning, the application must be carried out with the utmost care. Never alter the adjustment made by the doctor/physiotherapist/ or orthopedic technician. The responsibilities of the manufacturer shall lapse in case of inappropriate use or adaptation. The orthosis is made and to be used by only one patient. Should the orthosis be inappropriately used, the manufacturer declines all responsibility, as provided by the regulation for medical devices. In hypersensitive individuals, direct contact with the skin may cause redness and irritation. In case of pain, swelling, welts, or any other adverse reaction, contact your doctor immediately, and in case of a particularly severe adverse event, report the incident to the manufacturer and the competent authority of your country. The effectiveness of the orthopedic product is only guaranteed when all the components are properly in use.

#### SELECTION/SIZE

Code	REF.1557K R - REF.1557K L		
Size	S	M	L
Palm width cm	< 9	9/11	11/13
Colour	beige		

#### MAINTENANCE

⌘ Do not bleach
⌘ No chemical cleaning
⌘ Do not iron
⌘ Do not tumble-dry
🧼 Instructions for washing: wipe with a sponge dipped in lukewarm water and mild soap. Dry with a cloth. Consult an orthopaedic technician for the replacement of worn parts.

Do not dispose of the product or any of its components into the environment.

#### INDICATIONS

- Designed to keep the hand in a functional position
- Provides hand and wrist support in case of flaccid or spastic flexion
- Central or peripheral neurological injuries
- Treatment of rheumatic deformity

#### CONTRAINDICATIONS

Currently no known

#### CHARACTERISTICS AND MATERIALS

- KYDEX® structure
- Evazote® internal pad
- Velcro® fastening system
- Modeling with immersion 1 minute in hot water (70°) or use heat gun

#### PUTTING ON THE APPLIANCE

- Mould the splint according to the indications with the help of a heat gun.
- Secure the forearm component with the Velcro straps.
- Finally, fasten the thumb and finger straps.

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